

  
**Good Samaritan  
PACS**

1. Click the Login Button.  
Please be patient as Java is loading.




2. Say Yes to 'install the correct version of Java to run pacs:' And please be patient this might take up to 2 minutes depending upon your connection speed.

3. Enter in your Network username and password at this prompt:




To log out click the Arrow through the Door button and say yes to log out when the small dialog box pops up. At this point feel free to search by any field and use the PACS cheat sheet for help on the PACS Help Website.

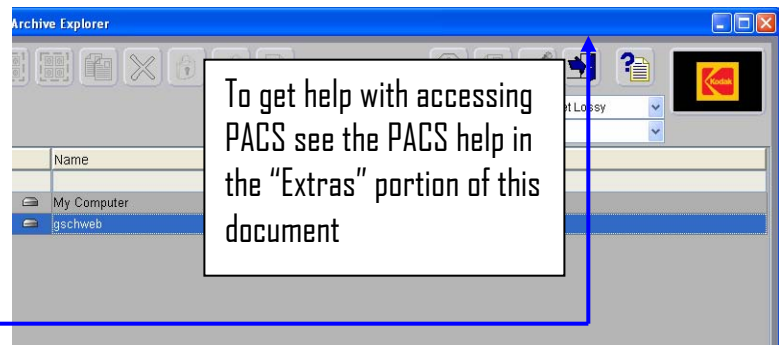





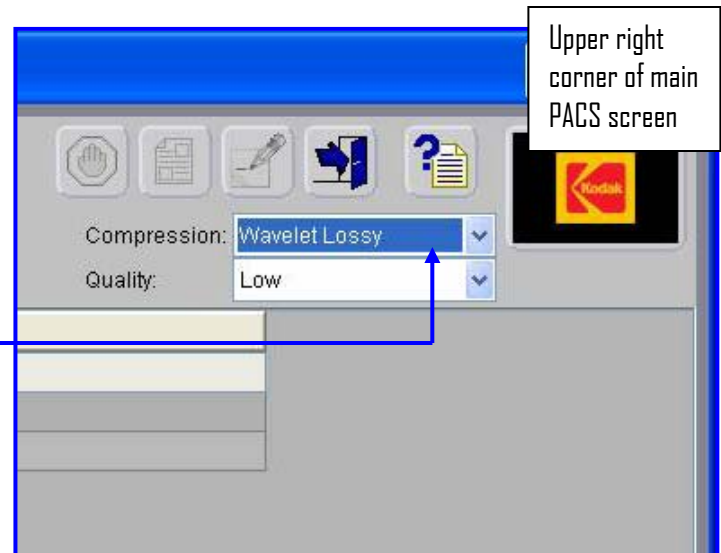



## PACS Settings

1. When using PACS remotely you will need to change the compression setting to "Wavelet Lossy" instead of "Uncompressed"


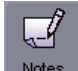





## Help Information

### Tool Tabs

-  **Edit** The Edit tab contains the tools that allow you to zoom, pan, rotate and flip the image.
-  **W/L** This tab represents the windowing tool. It lets you adjust the windowing values of the selected images.
-  **Layout** The tools in this tab let you configure the format in which groups and images are displayed.
-  **G/M** This tab has the tools that allow you to add and manipulate graphic and text annotations on images.
-  **Tools** The tools tab allows you to show the reference lines in images and view images in Cine mode.
-  **Conf** Conferencing lets two users view and discuss the same series of images.
-  **File** This tab has the tools that enable you to save the file locally and in multiple formats.
-  **My Tab** Lets you configure a user defined tab that contains your most useful tools.

### Other useful icons

-  **Reports** This icon directly links you to the radiology report for this exam as it is stored in PACS.
-  **Notes** This icon brings up "sticky notes." These are notes left by other users as quick reference information.
-  **Organize** This icon brings up the "mini-archive" which contains the complete list of studies for that patient.
-  **Done** Use this icon to signify that you are done with the current study and return to the Archive Explorer
-  **?** This icon will bring up the extensive help menu.

At any time if you are unsure of what tool the icon on the screen represents let the mouse pointer hover over the icon for a brief second or two and it the name will appear in a small pop-up.

## Logging Out

- 1.) To log out of the PACS web software Click Exit.
- 2.) In the Viewer Software the icon looks like the first one below and in the Archive Explorer the icon looks like the



second icon below

- 3.) The following window will appear.
- 4.) Click Yes to log out and return to the PACS homepage.



---

For questions or comments:  
The IT Help Desk is available from 7 am to 4:30pm  
By phone at (253) 697- 1506 or email at [helpdesk@goodsamhealth.org](mailto:helpdesk@goodsamhealth.org)